# Gastro<sup>+</sup> Gastrolyzer<sup>®</sup>

## Quick Start Guide

CE  
2797

Download  
your **FREE**  
patient management  
software  
**GastroCHART™**  
at...  
[www.bedfont.com](http://www.bedfont.com)

**Rx**only

  
**bedfont**<sup>®</sup>  
est. 1976

## Contents

English (EN).....	Page 4
Français (FR).....	Page 5
Deutsch (DE).....	Page 6
Español (ES).....	Page 7
Italiano (IT).....	Page 8
Português (PT).....	Page 9
Nederlands (NL).....	Page 10
Polski (PL).....	Page 11
Svenska (SE).....	Page 12
Suomi (FI).....	Page 13
日本(JP).....	Page 14
Русский (RU).....	Page 15
Magyar (HU).....	Page 16
Português Brasileiro (PTBR).....	Page 17

## Taking a Breath Test

1. Attach a breath sampling D-piece™ and a new SteriBreath™ Eco mouthpiece.
2. Turn on the device by pressing the power button once.
3. Press 'Breath Test' symbol on screen .
4. To cancel the breath test, press .
5. Inhale and hold breath for the pre-set 15 second countdown.
6. A beep will sound during the last 3 seconds of the countdown.
7. Blow slowly into mouthpiece, aiming to empty the lungs completely.
8. The ppm levels will rise and hold on-screen.
9. When the test is finished    will appear at the bottom of the screen.
10. To repeat the breath test, press  and repeat steps 4-8.
11. To return to the home screen, press .
12. To save the reading, press  and select the relevant patient profile.
13. Remove the D-piece™ between tests to purge sensor with fresh air.
14. To switch off, press and hold the power button for 3 seconds, device will also power off after 45 minutes of inactivity to save power.

## Taking a Facemask Test

1. Attach a new facemask and face mask sampling system.
2. Turn on the device by pressing the power button once.
3. Press .
4. To cancel the test, press .
5. Inhale and exhale in to the face mask, the device will take the reading in real-time.
6. The ppm levels will rise and hold at the peak level, when this has stopped rising for a period of 5 consecutive seconds the test result can be recorded and the test stopped.
7. When the test is finished    will appear at the bottom of the screen.
8. To repeat the test, press  once to return to the home screen and repeat steps 3-6.
9. Remove the face mask sampling system between tests to purge sensor with fresh air.
10. To switch off, press and hold the power button for 3 seconds, device will also power off after 45 minutes of inactivity to save power.

To download the full Gastro+™ manual and GASTROCHART™ please go to [www.bedfont.com](http://www.bedfont.com)

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© Bedfont® Scientific Limited 2024
**Bedfont® Scientific Ltd.**
Bedfont® Scientific Limited reserves the right to change or update this literature without prior notice.