

Screen for carbon monoxide poisoning instantly to help safeguard your staff with the **ToxCO[®]** range.



CE
2797



Saving lives, one breath at a time.

www.bedfont.com



Contents

| | |
|--|--------|
| What is Carbon Monoxide (CO) poisoning?..... | Page 3 |
| Diagnosing CO poisoning..... | Page 3 |
| Staff safety..... | Page 4 |
| ToxCO®..... | Page 5 |
| Features and Benefits..... | Page 6 |
| Technical specification..... | Page 6 |
| Consumables..... | Page 7 |
| References..... | Page 7 |

What is Carbon Monoxide (CO) poisoning?

CO poisoning occurs after the inhalation of CO¹. It is a significantly toxic gas, but being colourless, odourless, tasteless, and non-irritating, it is very difficult for people to detect². CO is a product of incomplete combustion and is often produced in domestic or industrial settings by vehicles and other gasoline-powered tools, such as heaters and cooking equipment. As a result, CO poisoning is the most common type of fatal poisoning in many countries³.

CO is and has been the most common cause of both accidental toxic poisoning and death in the United States for over 100 years⁴.

Diagnosing CO poisoning

One of the significant problems with diagnosing CO poisoning is that it can engender similar symptoms to flu⁵. Symptoms of CO poisoning include:

- Headache
- Nausea
- Vomiting
- Fatigue
- Dizziness

In the UK, the House of Commons All Parliamentary Gas Safety Group recommends providing medical professionals, such as ambulance personnel, on-call GPs and health visitors, with hand held devices so that they can test for exposure there and then⁵.



Staff safety

The ToxCO® has the capability of not only testing the level of CO on a patient's breath, but can also measure ambient levels of CO, helping to safeguard those entering a potentially dangerous incident.

In 2009, the London Ambulance Service (LAS) undertook a feasibility study into pre-hospital CO monitoring of patients. According to the study, previous incidents demonstrated that on many occasions ambulance personnel had been exposed to 'unidentified situations' of elevated CO concentration in which crews performed clinical assessment and treatment of patients. In the cases reviewed, the attending ambulance personnel neither had the equipment nor the protocols to screen CO as the presenting medical condition⁶.

Following on from the study, LAS have recommended CO devices for staff safety and to aid rapid patient screening.



Instantly screen for CO poisoning to help safeguard your staff.



ToxCO®

Instant screening for carbon monoxide poisoning.



Patient breath and atmospheric CO device



For use with adults



To alert staff of high atmospheric CO



For use with unconscious patients

Ideal for:

Ambulance services, first responders, GPs and health visitors.

The ToxCO® is an easy to use, non-invasive, breath/ambient CO device with three sampling modes. Breath samples can be taken from patients to screen for CO poisoning and also monitor ambient levels to alert staff of potentially fatal levels of CO in the air.

Features and Benefits

- Easy to use interface
- Maintenance reminders
- Tag readings
- Antimicrobial technology
- Large touchscreen
- Ambient monitoring
- Face mask breath testing

Technical Specification

| | |
|---|--|
| Breath test Concentration range (CO) | 0-50%COHb/0-500ppm |
| Face mask test Concentration range (CO) | 0-28%COHb/0-200ppm |
| Ambient Test Concentration range (CO) | 0-500ppm |
| Display | Full colour touchscreen |
| Detection principle | Electrochemical sensor |
| Repeatability | ≤±5% difference on consecutive readings |
| Accuracy | ≤±3ppm/10% - whichever is greater* |
| Power | 3 x AA (LR6 or equivalent) - up to 1000 minutes 1 x CR2032 lithium coin cell |
| T ₉₀ response time | <30 seconds |
| Operating temperature | 0-45°C |
| Storage/ transport temperature | 0-50°C |
| Operating/storage/transport pressure | Atmospheric ±10% (912-1114mbar) |
| Operating humidity | 15-90% non-condensing |
| Storage/transport humidity | 5-95% |
| Sensor operating life | 2 years |
| Sensor sensitivity | 1ppm |
| Sensor drift | <5% per annum |
| Dimensions | Approx. 37 x 77 x 140mm |
| Weight | Approx. 215g (including batteries) |
| Materials | Case: polycarbonate/ABS blend with Antimicrobial additive D-piece™: polypropylene SteriBreath™ Eco: paper OneBreath™: polypropylene |
| H ₂ cross interference | ≤6% |

*Readings of >500ppm at temperatures between 0-14°C can decrease accuracy to ≤±3ppm/15%.


Consumables

SteriBreath™ Eco: The SteriBreath™ Eco mouthpieces come in boxes of 200 mouthpieces, and each one is individually wrapped to ensure optimum infection control. The SteriBreath™ Eco mouthpiece is entirely made from paper and therefore it is 100% recyclable and 100% biodegradable, including its packaging. Even better, all the materials are from sustainable sources.



Order code: STERIBREATH-ECO (200 per box)

D-piece™: The D-piece™ is used to attach the SteriBreath™ Eco to the device. The D-piece™ incorporates a one-way valve and an infection control filter, which are proven to remove and trap >99% of airborne bacteria⁶. The D-Piece™ should be changed every four weeks, or more often if visibly soiled. An automatic reminder will appear on the screen every 28 days.

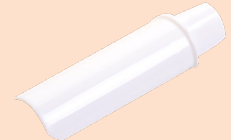

Order code: D-PIECE-3 (12 per box)

Face mask sampling system: This sampling system is single patient use and allows the patient to breathe normally through a face mask in order to produce a breath sample.

Order code: ISSA-V-2 (1 sampling kit)

Small face mask

Order code: EC60-IM-V

Medium face mask

Order code: EC60-MM-V

Large face mask

Order code: EC60-AM-V



References

1. Ernst, A. and Zibrak, J. D. (1998) 'Carbon monoxide poisoning', New England Journal of Medicine, 339(22), pp. 1603–1608. Guzman, J. A. (2012) 'Carbon monoxide poisoning', Critical Care Clinics, 28(4), pp. 537–548.
2. House of Commons All Parliamentary Gas Safety Group. (2009) 'Raising Medical Professionals Awareness of Carbon Monoxide Poisoning'.
3. Humber, A. (2009) 'A Feasibility study into the prehospital carbon monoxide poisoning of patients', London Ambulance Service NHS.
4. Omaye, S. T. (2002) 'Metabolic modulation of carbon monoxide toxicity', Toxicology, 180(2), pp. 139–150.
5. Wright, J. (2002) 'Chronic and occult carbon monoxide poisoning: We don't know what we're missing', Emergency Medicine Journal, 19(5), pp. 386–390.
6. Public Health England. An Evaluation of Filtration Efficiencies Against Bacterial and Viral Aerosol Challenges Report No. 17/001. London: Public Health England; 2017.



Contact Bedfont® or one of our worldwide **ToxCO®** distributors for a free demonstration.

www.bedfont.com
Tel: +44 (0)1622 851122
Email: ask@bedfont.com

A full list of our worldwide distributors can be found at
www.bedfont.com/distributors/

Our family, innovating health, for yours.

Visit www.bedfont.com/resources to view this document in other languages.



Bedfont® Scientific Ltd.
Station Road, Harrietsham, Maidstone,
Kent, ME17 1JA England
Tel: +44 (0)1622 851122 Fax: +44 (0)1622 854860
Email: ask@bedfont.com Web: www.bedfont.com



Emergo Europe B.V.
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands.

© Bedfont® Scientific Limited 2024

Issue 25 - September 2024, Part No: MKT022_USA
Bedfont® Scientific Limited reserves the right to change or update his literature without prior notice. Registered in: England and Wales. Registered No: 1289798



MD 502905