




iCOquit[®] Smokerlyzer[®]

Single-Patient-Use CO device, to help Mums to Be quit smoking remotely,
one breath at a time.

The iCOquit[®] is a personal Bluetooth[®] carbon monoxide (CO) device belonging to the well-known Smokerlyzer[®] range of stop smoking devices. Used alongside the free iCOquit[®] Mums to Be app, this personal CO device enables patients to safely and easily monitor their CO readings remotely and share their results and progress with their midwife. Portable, pocket-size, and with instant results, the iCOquit[®] is ideal for remote CO monitoring.

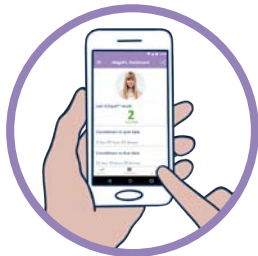

Visual motivation to help you
stop smoking

Features and Benefits

Connects to a
smartphone via
Bluetooth[®]



Patients can easily
share their progress
with Health Care
Professionals



Monitor your quit
smoking progress



Use anytime,
anywhere



Quick breath test,
instant results



Become smoke-free



It couldn't be easier to quit smoking with the iCOquit® Smokerlyzer®



Select breath test on the dashboard.




Connect the iCOquit® device.



Inhale and hold your breath for 15 seconds.



Slowly exhale in to the iCOquit®.

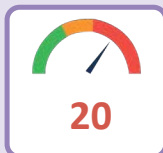

See your results instantly.



Repeat until you're smokefree.

Technical Specification

Gas Measured	Carbon Monoxide (CO)
Detection Principle	Electrochemical
Concentration range	0 - 100 ppm
Monitoring resolution	1 ppm
Monitoring accuracy	± 3 ppm/ $\pm 10\%$ of reading*
Monitoring repeatability	± 3 ppm/ 5% (within 120 sec)*
Operating lifetime	12 months
Display	Via the iCOquit® mobile application
Battery Life	12 months and battery is non-replaceable
Power	3 v Lithium-ion coin cell
T ₉₀ response time	< 30 Seconds
Operating Temperature	15 - 35°C
Storage/Transport Temperature	0 - 50°C
Operating & Storage/Transport Humidity	10 - 90% RH non-condensing
Operating & Storage/Transport Pressure	912.0 - 1114.0 mbar
Dimensions	Less than 150 mm long x 30 mm diameter
Weight	Less than 100 g
Materials	Plastic casing: ABS and Polycarbonate
H ₂ Cross Sensitivity	$\leq 6\%$
Time required to warm/cool from the minimum/maximum storage temperature before use	15 minutes

*whichever is greater

The iCOquit® is also ideal for:

- Health professionals
- Pharmaceutical companies
- App developers



Visit www.bedfont.com/resources to view this document in other languages.



Bedfont® Scientific Ltd.
 Station Road, Harrietsham, Maidstone,
 Kent, ME17 1JA England
 Tel: +44 (0)1622 851122 Fax: +44 (0)1622 854860
 Email: ask@bedfont.com Web: www.bedfont.com



Emergo Europe B.V.
 Westervoortsedijk 60
 6827 AT Arnhem
 The Netherlands.

© Bedfont® Scientific Limited 2024

Issue 8 - September 2024, Part No: MKT523_MAT

Bedfont® Scientific Limited reserves the right to change or update his literature without prior notice. Registered in: England and Wales. Registered No: 1289798

