GastroCH₄ECK[®] Gastrollyzer[®]

Hydrogen and methane monitoring to aid in the diagnosis of gastrointestinal disorders.

Quick Start Guide

Contents

English (EN)	4
Français (FR)	8
Deutsch (DE).....	12
Español (ES)	16
Italiano (IT)	20
Nederlands (NL).....	24
Português (PT)	28
Polski (PL)	32
Svenska (SE).....	36
հայերէն (AM)	40
български (BG)	44
Dansk (DK)	48
Suomi (FI)	52
Γραϊκόζ (GR)	56
Hrvat (HR).....	60
Magyar (HU)	64
日本 (JP)	68
Lietuvių (LT)	72

Norsk (NO).....	76
Русский (RU)	80
Slovenski (SI).....	84
Slovenský.....	88
Türkçe (TR).....	92
Україна (UA).....	96

GastroCH₄ECK®

Cautions

- **CAUTION:** DO NOT switch on for the first 24 hours. After 24 hours, switch the device on and allow ≤ 2 minutes for the GastroCH₄ECK® Gastrolyzer® to start up.
- **CAUTION:** Ensure the device is used within the stated operating temperature and humidity ranges. Operating temperature is 15 - 35°C. Operating humidity is 15 - 90% RH (non-condensing).
- **CAUTION:** Never use alcohol, substances containing alcohol, or other organic solvents as these vapours will damage the sensors.
- The GastroCH₄ECK® should be switched off once every 24 hours.

Consult instructions for use for Installation and Set-up.

www.bedfont.com

Installation

On receipt of a new GastroCH₄ECK®:

1. Remove the unit from the packaging supplied.
2. Place the GastroCH₄ECK® on a level surface with a 30 cm (1 ft) clearance envelope for ventilation.
3. DO NOT switch on for the first 24 hours.
4. After 24 hours, switch the device on and allow ≤ 2 minutes for the GastroCH₄ECK® to start up.
5. Please allow the GastroCH₄ECK® to be switched on for at least 5 minutes prior to calibration.

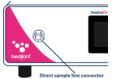
Calibrating GastroCH₄ECK®

NOTE: The device should be calibrated prior to first use, after transportation, and every 4 weeks.

NOTE: The GastroCH₄ECK® Gastrolyzer® device temperature needs to be at 21°C ± 4°C to be able to perform a calibration.

1. Make sure the power cord is connected and the GastroCH₄ECK® is switched on.
2. Open the settings menu by pressing , and start the calibration process by choosing the calibrate icon .
3.  will be shown onscreen while the sensor is zeroing. This will happen automatically. **NOTE:** Do not connect the gas at this stage.
4. Once the zero has complete, follow the calibration process as shown onscreen.
5. Verify calibration gas is the correct gas concentration, 100 ppm CH₄, 100 ppm H₂, 20.9% and O₂. Press  to continue.
6. When prompted by , Screw the gas canister into the regulator. Press  to continue.
7. When prompted by , attach the CaliAdaptor to device via the sample line and open the gas canister. Press  to begin the calibration.
8.  will be shown onscreen while the sensor is zeroing.
NOTE: Gas flow should be set to 1 litre per minute.
9. A successful calibration will be indicated by , press  to return to the home screen.
10. A failed calibration will be indicated by ; press  to attempt calibration again. If the problem persists, see the 'Troubleshooting' section of the user manual. If failure persists contact Bedfont® or their distributor for support.

Taking a mouthpiece breath test

1. Make sure the power cord is connected and the GastroCH₄ECK® is switched on.
2. Connect the sample line to the direct sample line connector; a click should be audible.
3. Attach the mouthpiece to the sample line .
4. Press .
5.  will be shown onscreen while the sensor is zeroing.
6. When prompted by , inhale and hold breath for the pre-set 15 seconds.
7. A beep will sound during the last 3 seconds of the countdown.
8. Blow slowly into the mouthpiece, aiming to keep the arrow in the green section .
9. The arrow will change colour as the O₂ level in the breath sample reduces to the target 15%, at which point it will turn green and the test will automatically stop after 3 seconds .
10. If you cannot complete the test, press .
11. The final results will be shown onscreen.
12. Press  for a visual indicator of the results:

 = good  = satisfactory  = poor, retake test

13. Press  to return to the results.
14. To take another test, press  and then repeat steps 4 - 13.
15. Remove the mouthpiece and sample line in between tests to purge the sensors with fresh air.

Collect a breath bag sample

1. Remove the blue plugs from the breath bag.
2. Place two clamps on the inlet and outlet but leave them open .
3. Attach a mouthpiece to the inlet .
4. Exhale gently into the bag until your lungs are nearly empty.
5. When your lungs are nearly empty, close the clamp at the outlet .
6. Fill the bag and then close the clamp at the inlet .
7. Replace the blue plugs.
8. To take multiple breath bag samples, repeat steps 1-7 with new breath bags.

Analyse breath bag sample

1. Make sure the power cord is connected and the GastroCH₄ECK® is switched on.
2. Attach the moisture filter to the bag sample connector .
3. Press .
4. Remove the blue plug from one end of the breath bag and attach to the moisture filter. Open the clamp on the inlet of the breath bag.
5. Press  to begin breath bag sampling.
6.  will be shown onscreen while the sensor is zeroing.
7. Once zeroed, the GastroCH₄ECK® will automatically begin bag sampling for 45 seconds as indicated by  make sure that the clamp is open.
8. The final results will be shown onscreen.
9. Press  for a visual indicator of the results:

 = good  = satisfactory  = poor, retake test

10. Press  to return to the results.
11. To take another test, press  and then repeat steps 3-10.

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© Bedfont® Scientific Limited 2025
Bedfont® Scientific Ltd.
Emergo Europe B.V.
Bedfont® Scientific Limited reserves the right to change or update this literature without prior notice.