FeNOchart™

User Manual

CE
2797

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Manuale d'uso

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Handleiding

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***NObreath® software, to download, store
and analyse data.***

LANGUAGES

Contents

English	3
Français	26
Deutsch	49
Español	72
Italiano	95
Nederlands	118
Português	141
Polski	164
Svenska	187
български	210
Dansk.....	233
Γραικός	256
한국인.....	279
Русский	302
Türkçe	325

Definitions

WARNING: Indicates a potentially hazardous situation, which, if not avoided, may result in minor or moderate injury.

CAUTION: Indicates a potentially hazardous situation, which, if not avoided, may result in damage to the device.

NOTE: Used to call attention to notable information that should be followed during use.

Contents

Definitions	3
Welcome to FeNOchart™	5
Intended Use	5
Minimum System Requirements	5
Installation and Set-up	6
User Interface	8
<i>User Accounts</i>	9
<i>Create a New User Account</i>	9
<i>Reset a User Account</i>	10
<i>Delete a User Account</i>	10
Patient Profiles	11
<i>Patient Database</i>	11
<i>Adding a Patient Profile</i>	11
<i>Editing a User Account</i>	12
<i>Deleting a Patient Profile</i>	12
<i>Take a Test in a Patient Profile</i>	13
<i>Searching Patient Profiles</i>	13
Synchronisation with FeNOchart™	14
<i>Download results to an existing profile</i>	15
<i>Deleting an existing profile via synchronisation</i>	16
<i>Clear readings from an existing patient profile via synchronisation</i>	17
<i>Create a new patient profile in FeNOchart™ via synchronisation</i>	17
<i>Replace results on an existing profile via synchronisation</i>	18
How-To Videos	19
Taking a Breath Test	19
Generating Reports	21
Ambient Testing	21
Device Options	22
Software Settings	22
<i>FeNOchart™ Settings</i>	23
<i>Edit User Account</i>	23
<i>Firmware Update Tool</i>	23
<i>Help Menu</i>	23
Icons Explained	24
Responsible Manufacturer and Contacts	25

Welcome to FeNOchart™

FeNOchart™ is a free, patient management software for use with the NObreath® FeNO (Fractional exhaled Nitric Oxide) device. This manual outlines the basic functionality of the FeNOchart™ software, and provides a guide for how to use each function.

NOTE: *Only technical data and no patient data is collected by Bedfont®.*

Intended Use

FeNOchart™ is a standalone piece of software, which is used with the NObreath® device. The software is used as an aid for healthcare professionals to record patient FeNO results.

The FeNOchart™ software will be able to create a graph from the patient results. The healthcare professional will be able to generate a report from the patient profile data.

The FeNOchart™ information including patient results and graphs will be able to be exported into a PDF format.

The FeNOchart™ software can be connected via USB to the NObreath®.

It is intended to be used by healthcare professionals in medical institutions.

It is recommended that this software is downloaded and installed on a stand-alone computer, not connected to a network, to ensure optimum patient data security. If this software will be installed on a shared network, please ensure that both a domain account with a secure password has been set up to protect patient data.

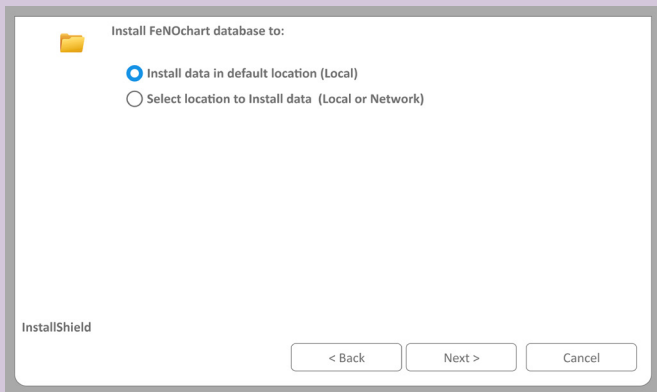
Minimum System Requirements

Operating System:	Windows 7
Processor:	1 GHz
RAM:	512 MB
Disk space	
32-bit:	4.5 GB
64-bit:	4.5 GB
Connection:	USB or Bluetooth®

Installation and Set-up

FeNOchart™ can be downloaded from www.nobreathfeno.com

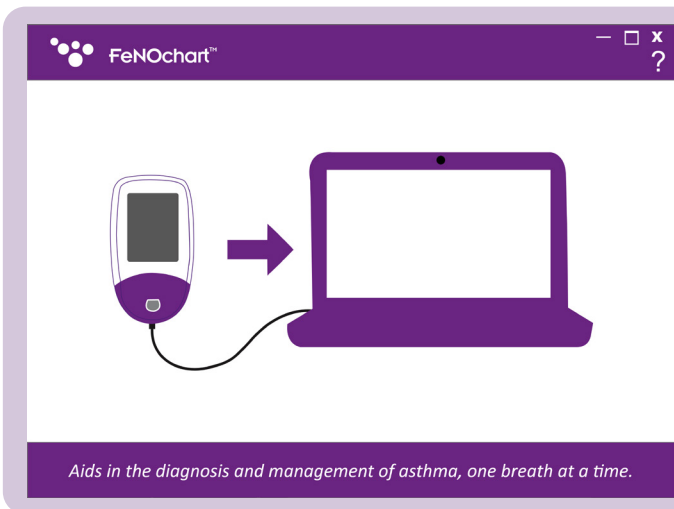
It is recommended that this software is downloaded and installed on a stand-alone computer, not connected to a network, to ensure optimum patient data security. If this software will be installed on a shared network, please ensure that both a domain account and a Bedfont® product account with a secure password has been set up to protect patient data.

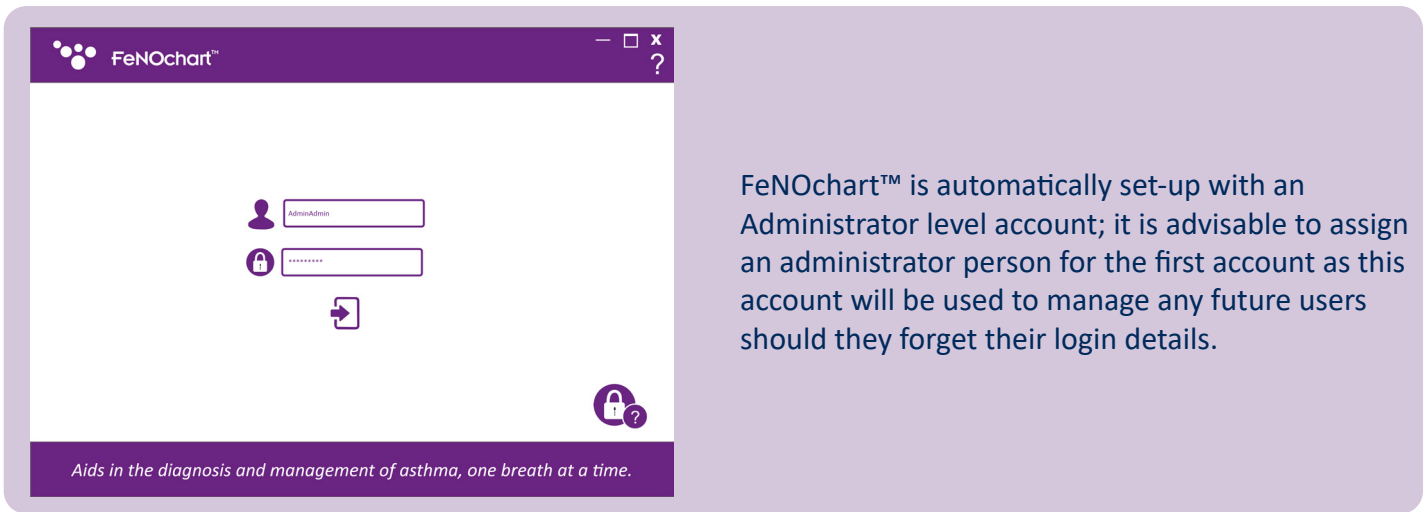
Follow the instructions on-screen to complete the setup.

During the initial install of FeNOchart™, there will be an option of saving FeNOchart™ in the default location (locally) or choice of location (locally or on a network).

Once FeNOchart™ is fully installed, a prompt will appear to connect the NObreath® to the computer in the first instance.

FeNOchart™ will register the connected device and an alert will appear once a device has been connected. A prompt will then ask for a database to be created. Follow the onscreen instructions to set-up the database. Once the database has been created, the login page will appear.

The Administrator level account has pre-set log-in details:

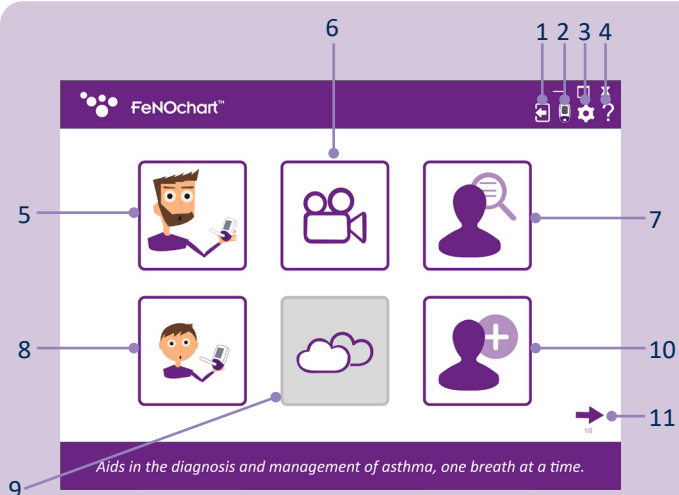
Username: AdminAdmin

Password: Password

Upon the first login, it is advised to change the pre-set username and password, as listed above, and also to provide 3 security questions and answers. These will be required to reset an account if a password is forgotten.

Once the administrator level account has been set up, the administrator will then be able to set up the other user accounts. Please see the 'User Accounts' section of this manual for more information.

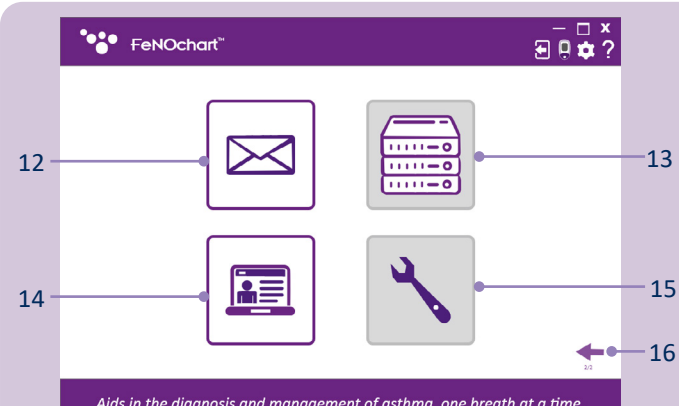
User Interface

The screenshot shows the FeNOchart Home Screen. At the top, there is a purple header with the FeNOchart logo and a navigation bar containing icons for mobile, settings, and help. Below the header, there are six main icons arranged in a 2x3 grid. The icons are: 1. Log out (person with X), 2. NObreath® options (person with NObreath logo), 3. Settings (gear), 4. Help (question mark), 5. Adult breath test (adult with phone), 6. Demo videos (video camera), 7. Patient database (person with magnifying glass), 8. Child breath test (child with phone), 9. Ambient test (coming soon) (clouds), 10. Add a new patient (person with plus sign), and 11. Next page (arrow). At the bottom, there is a purple footer with the text "Aids in the diagnosis and management of asthma, one breath at a time."

Home Screen

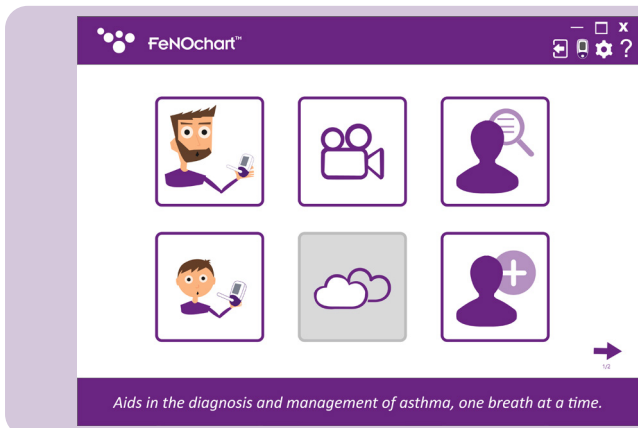
1. Log out
2. NObreath® options
3. Settings
4. Help
5. Adult breath test
6. Demo videos
7. Patient database
8. Child breath test
9. Ambient test (coming soon)
10. Add a new patient
11. Next page

The screenshot shows the FeNOchart settings screen. At the top, there is a purple header with the FeNOchart logo and a navigation bar containing icons for mobile, settings, and help. Below the header, there are four main icons arranged in a 2x2 grid. The icons are: 12. Email settings (envelope), 13. Server settings (coming soon) (server rack), 14. Administrator Panel (person with laptop), and 15. Troubleshooting (coming soon) (wrench). At the bottom, there is a purple footer with the text "Aids in the diagnosis and management of asthma, one breath at a time."

12. Email settings
13. Server settings (coming soon)
14. Administrator Panel
15. Troubleshooting (coming soon)
16. Previous page

NOTE: A user level account cannot access the Administrator Panel to reset security questions or change account privileges; only an administrator account can access the administrator panel.

User Accounts

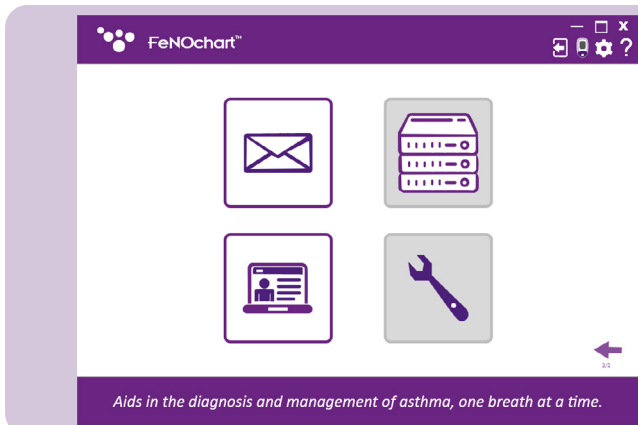
NOTE: User level accounts will only have access to the following features.

Account		
Patient Data	✓	✓
Management	✓	✗

There are 2 types of user accounts:

1. Administrator level
2. User level

NOTE: Only an administrator can create and delete user accounts.

Create a New User Account

To create a new user account, access the Administrator Panel from the 2nd page of the home screen. From here user accounts can be created or deleted, and passwords can be reset.

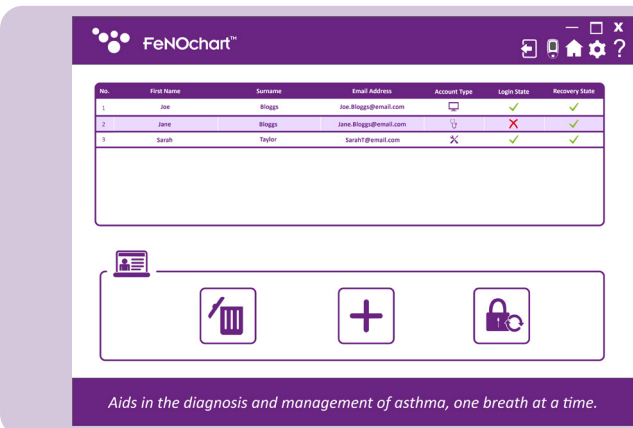
Click the '+' icon at the bottom of the screen to add a new user.

A screenshot of the FeNOchart user creation form. The interface features a purple header with the FeNOchart logo and navigation icons. Below the header is a form with the following fields: Account Type (dropdown menu), Username, First Name, Password, Surname, Confirm Password, and Email Address. A small table with account permissions is visible in the bottom left of the form area. A 'Save' icon is located at the bottom right of the form. At the bottom of the screen, a purple banner contains the text: "Aids in the diagnosis and management of asthma, one breath at a time."

Fill out the details required and click the 'Save' icon.

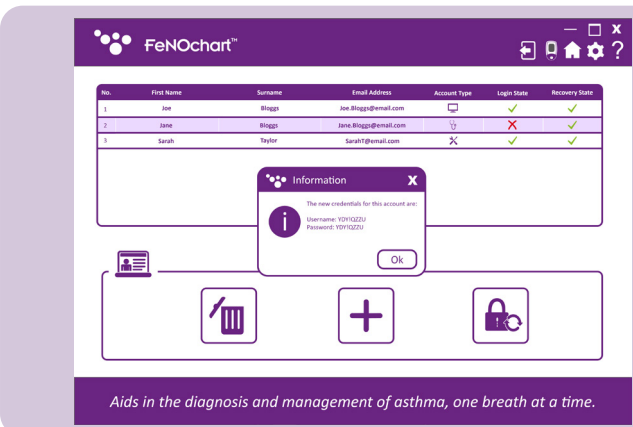
A prompt will appear asking for confirmation; click yes to complete the account set-up.

NOTE: Upon logging into a new account, all users will be required to change their password and set their security questions and answers.

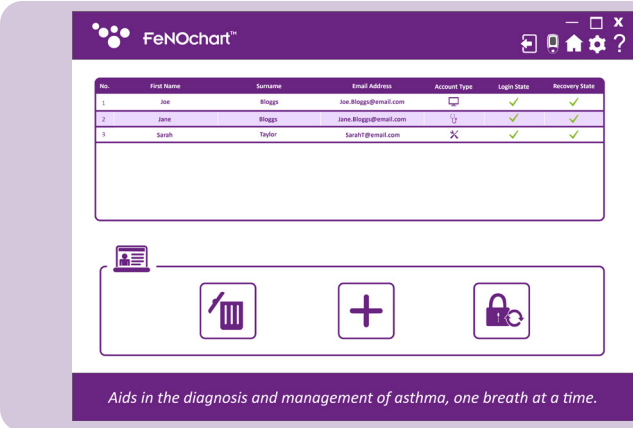
Reset a User Account

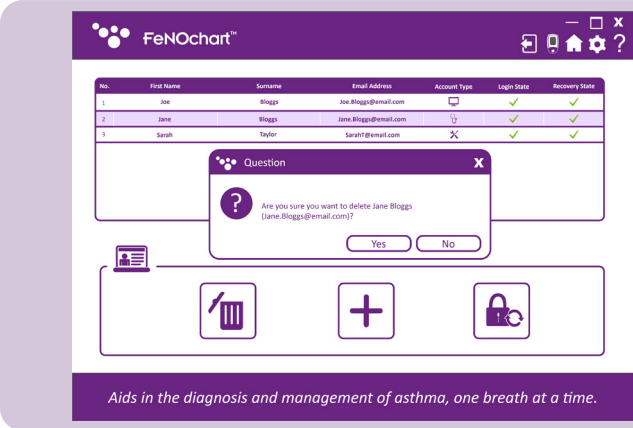
When a user has forgotten their password or is locked out of their account from failed recovery attempts, only an administrator can reset their account. The Administrator Panel shows the status of a user account; the red 'x' indicates an account which needs resetting.

Clicking the password reset button will generate a new log-in and password for the user to access their account.

Once the user has used the generated details to log in and reset their password, their recovery state will return to a green tick.

Delete a User Account

To delete a user account, select their profile in the Administrator panel and then click the 'Delete' icon. A prompt will appear asking for confirmation; click yes to delete the user account.

Patient Profiles

Patient Database

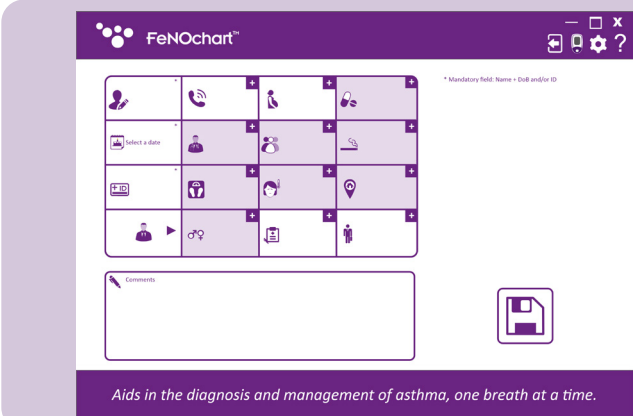
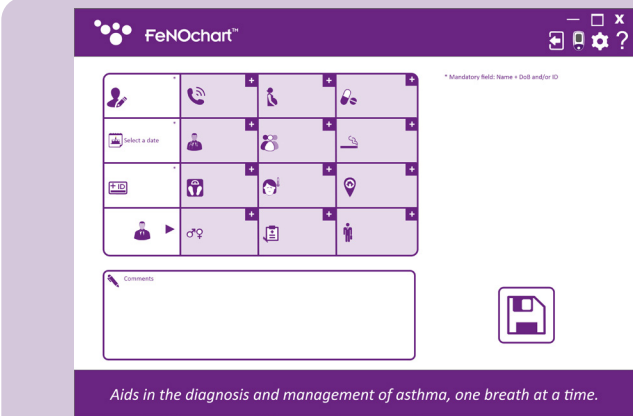
FeNOchart™ has the ability to create and store patient details, enabling a patient’s FeNO readings to be tracked over time. From the database, it is possible to edit patient details, view a list or graph of results, take and save a test directly to a profile, and export reports.

Adding a Patient Profile

To create a patient profile, click on the ‘Add a New Patient’ icon.

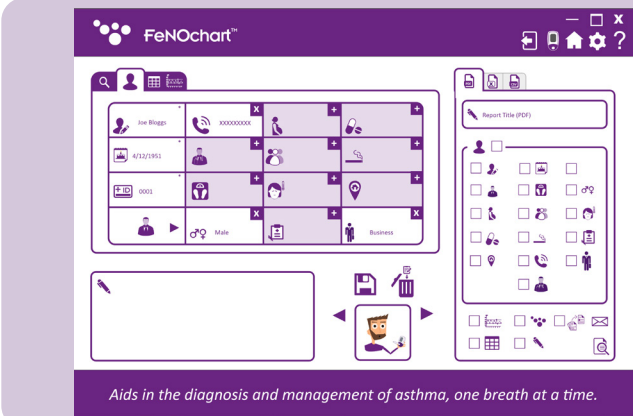
By default, only the mandatory fields are editable. If additional information is desired, there are 2 options:

1. Manually enable/disable the various criteria by clicking the ‘+’ icon.
2. Amend the default criteria in the settings menu (see the Patient Profile Settings section of this manual).

To manually add additional information, click on the ‘+’ icon at the top right of the criteria field; when enabled, the criteria box will become white instead of purple and the information can be entered. To disable, click the ‘x’ on the top right of the criteria field.

Enter the patient information; mandatory fields marked by an *. The date of birth field must also be before today, all other fields are optional.

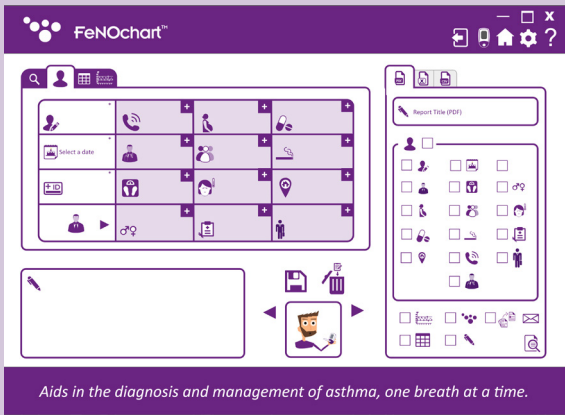
Click the ‘Save’ icon to save the profile to the FeNOchart™ database. A prompt will appear asking for confirmation; click yes to complete the patient set-up.

The patient can then be found in the Patient Database.

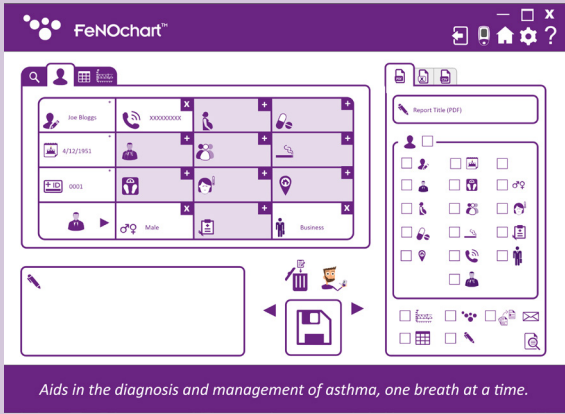
Editing a User Account

To edit a patient profile, open the patient database and select the patient to be modified. Click in the 'Profile' tab.

Edit the information as desired. To save the results, click the 'Arrows' to select the 'Save' icon.

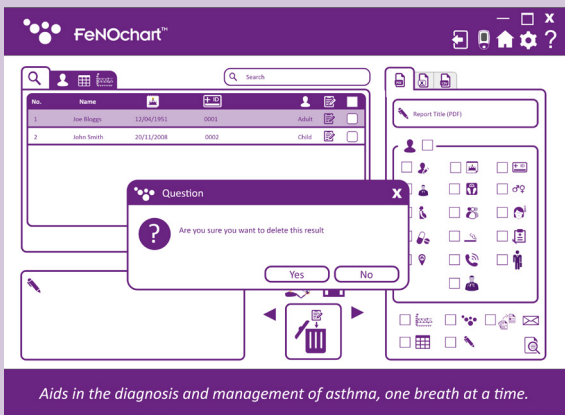
Click the 'Save' icon and a prompt will appear asking for confirmation; click yes to keep the changes.

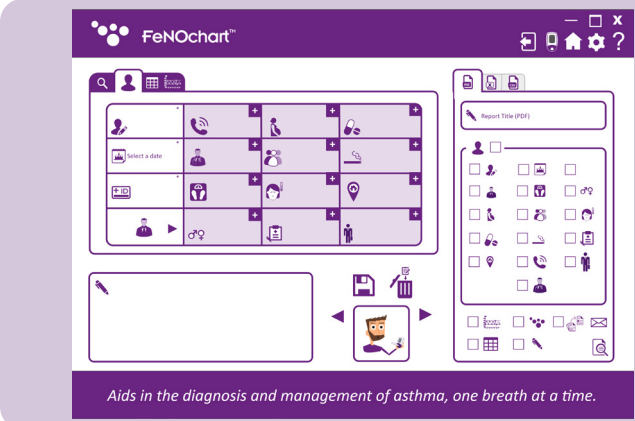
Deleting a Patient Profile

To delete a patient profile, open the patient database and select the patient to be deleted. Click the arrows to tab the action button to the 'Delete' icon.

Click the 'Delete' icon and a prompt will appear asking for confirmation; click yes to delete the patient profile.

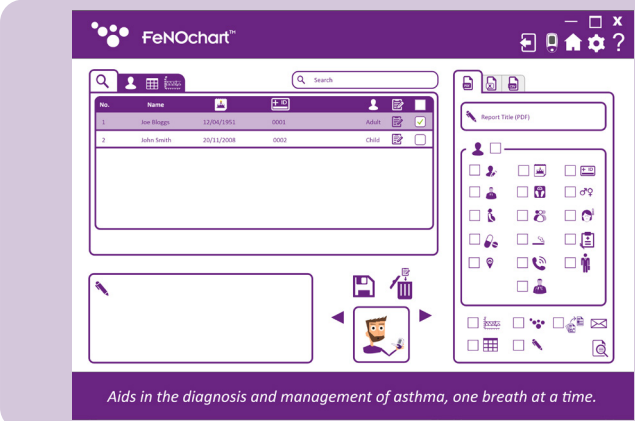
Take a Test in a Patient Profile

To start a test from a patient profile, open the patient database and select the patient to be tested. Click the arrows to select the action button to the 'Breath Test' icon.

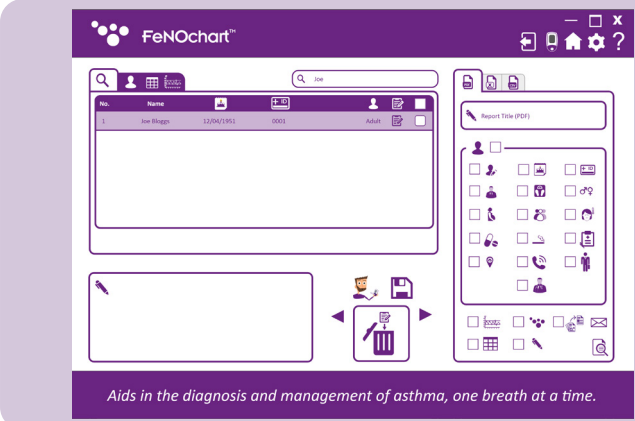
NOTE: The 'Breath Test' icon will either be an adult or a child depending on what their account type is.

Click the 'Breath Test' icon and a prompt will appear asking for confirmation to start the test; click yes to begin and follow the on-screen instructions to perform a NObreath® FeNO test. The results will automatically be saved to the patient's profile.

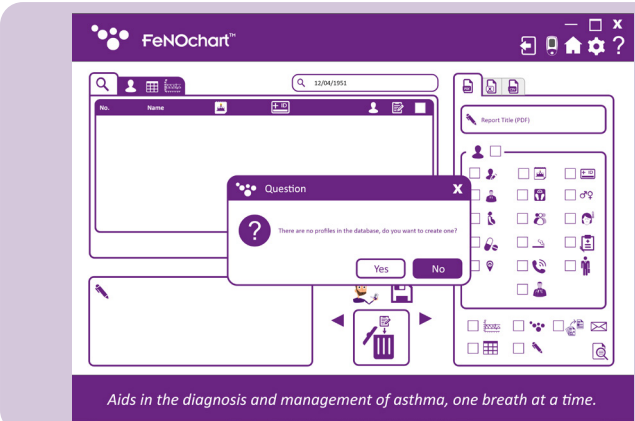
Searching Patient Profiles

To locate a specific patient in the patient database, use the search bar to enter either a patient name, ID or number.

When a patient name, ID or number has been entered into the search bar, only patients that include the searched criteria will appear on screen.

If the information entered into the search bar doesn't relate to any patient in the database, or isn't one of the criteria specified above, a pop-up will appear on screen, asking if a patient profile should be created using the information searched.

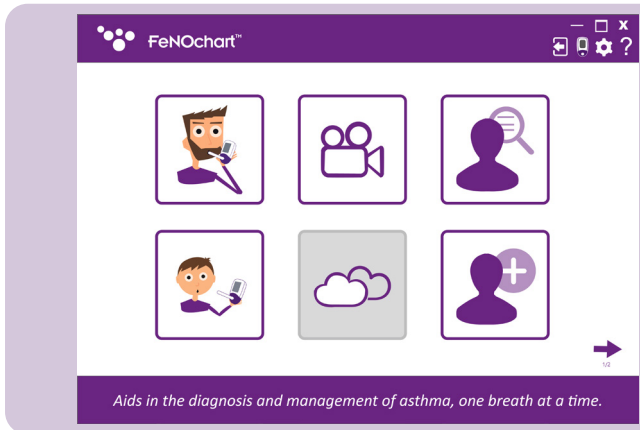
If this is required, select yes, or no to search again.

Synchronisation with FeNOchart™

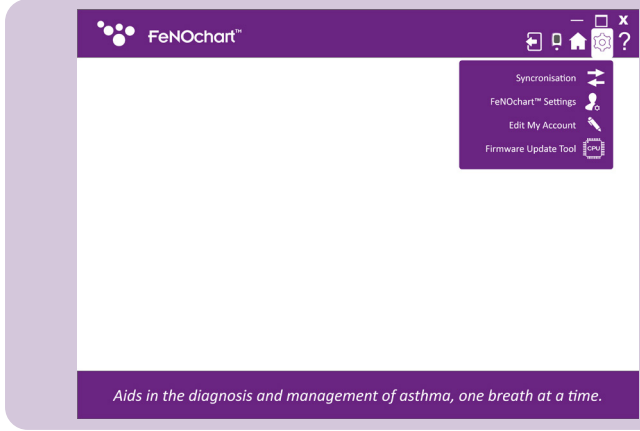
FeNOchart™ has the ability to download patient profiles and results from the NObreath® and store them in the FeNOchart™ database.

NOTE: Only technical data and no patient data is collected by Bedfont®.

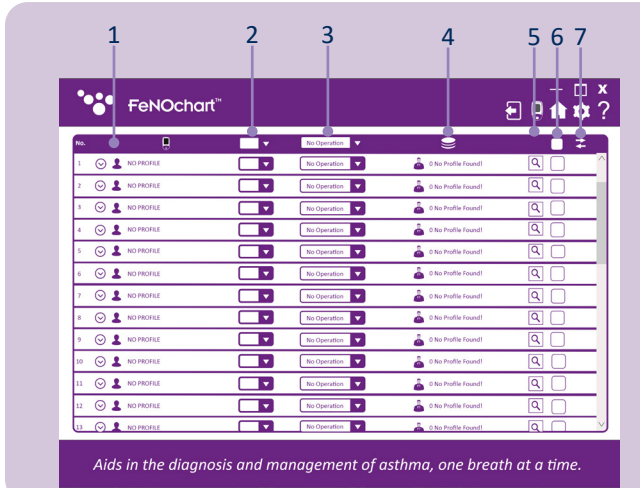
NOTE: Synchronisation will only be successful if readings exist on a profile on the NObreath®. If a profile exists but no readings are saved, synchronisation will not be possible as there is no data to synchronise.

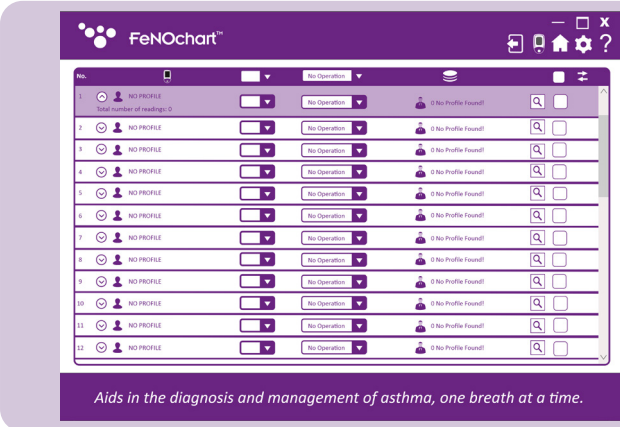
To begin synchronisation, first ensure the NObreath® FeNO device is switched on and connected to the computer. Once connected, a small NObreath® icon will appear in the top-right corner of FeNOchart™.

To access the synchronisation screen, click the 'Settings' icon and then 'Synchronisation'.

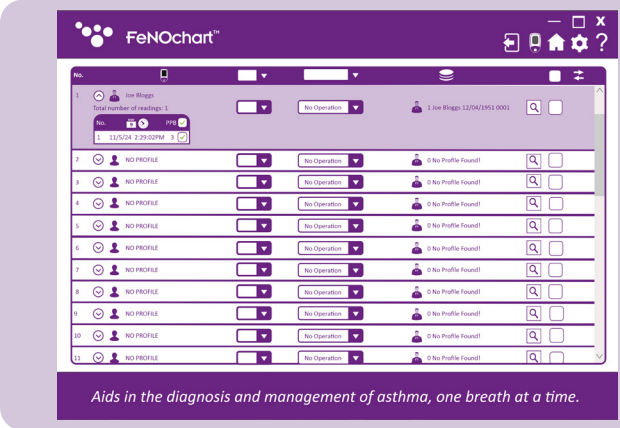
1. Patient profiles on the NObreath®
2. Synchronisation action
3. Enable/disable synchronisation
4. Patient profiles in the FeNOchart™ database
5. FeNOchart™ patient profile search
6. Overwrite patient name
7. Synchronise button

Download results to an existing profile

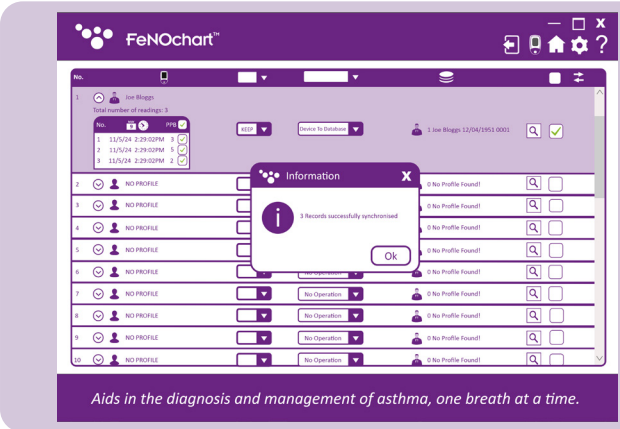
Click on the patient to be synchronised. By clicking on the arrow next to the patient name, it is also possible to select specific readings to synchronise with FeNOchart™.

Next, enable synchronisation by changing 'No Operation' to 'Device to Database'.

Ensure the 'KEEP' function is selected as this will download the results from the selected profile on the NObreath® and add them to an existing profile in FeNOchart™.

FeNOchart™ will automatically recognise patient names/IDs and suggest an existing profile to synchronise to if it recognises similar details. If the incorrect patient is suggested, click on the magnifying glass to search the FeNOchart™ patient database for alternative profiles.

A new window will open listing the existing patient profiles stored within the FeNOchart™ database. Select the correct patient from this list and click 'Select Profile' to continue.

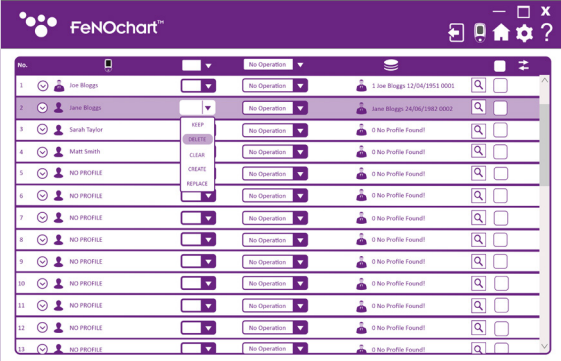
To complete the synchronisation, click the 'Synchronisation' button at the top right of the table. A pop-up will appear upon completion.

Deleting an existing profile via synchronisation

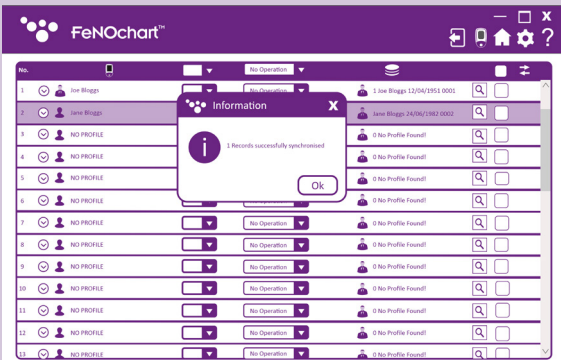
Select the patient to be deleted from the FeNOchart™ database.

NOTE: The suggested patient profile will be erased from the FeNOchart™ database only; the matching profile will remain on the NObreath® device.

Next, enable synchronisation by changing 'No Operation' to 'Device to Database'. Ensure the 'DELETE' function is selected as this will remove the matching profile from the FeNOchart™ database. Verify the suggested profile is the correct profile to be erased from the FeNOchart™ database or search the database for another profile.

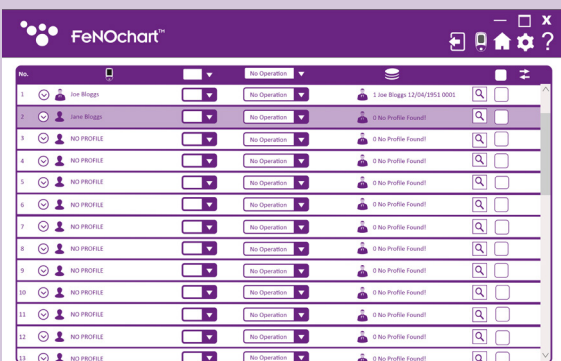
Aids in the diagnosis and management of asthma, one breath at a time.

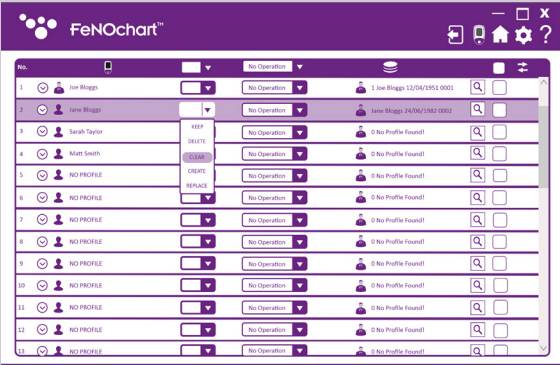
Aids in the diagnosis and management of asthma, one breath at a time.

To complete the synchronisation, click the 'Synchronisation' button at the top right of the table. A pop-up will appear upon completion.

Aids in the diagnosis and management of asthma, one breath at a time.

Click 'OK' to continue and the patient profile will have been erased from the FeNOchart™ database.

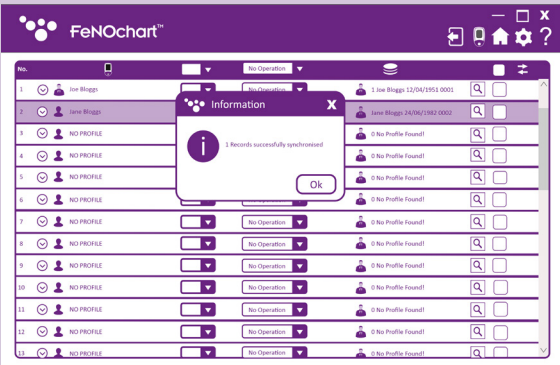
Aids in the diagnosis and management of asthma, one breath at a time.

Clear readings from an existing patient profile via synchronisation

Select the patient to clear results from, from the FeNOchart™ database.

Next, enable synchronisation by changing 'No Operation' to 'Device to Database'.

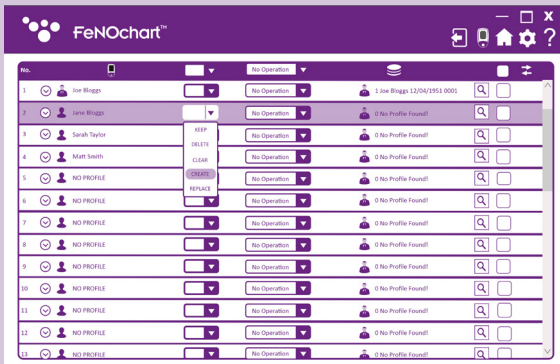
Ensure the 'CLEAR' function is selected as this will clear the results within the matching profile from the FeNOchart™ database. Verify the suggested profile is the correct profile to clear the results from or search the database for another profile.

Aids in the diagnosis and management of asthma, one breath at a time.

NOTE: The results from the suggested patient profile will be cleared from the FeNOchart™ database only; the results for the matching profile will remain on the NObreath® device.

To complete the synchronisation, click the 'Synchronisation' button at the top right of the table. A pop-up will appear upon completion. The patient profile will still exist within the FeNOchart™ database but will no longer have any results.

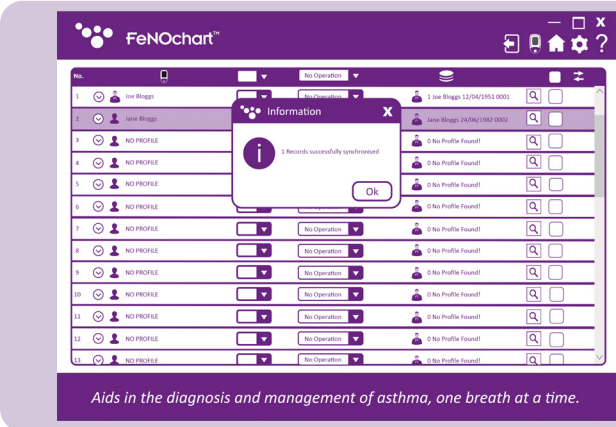
Aids in the diagnosis and management of asthma, one breath at a time.

Create a new patient profile in FeNOchart™ via synchronisation

Select the patient to be synchronised with the FeNOchart™ database. If FeNOchart™ cannot find/suggest an existing profile within the database to synchronise with, one can be created.

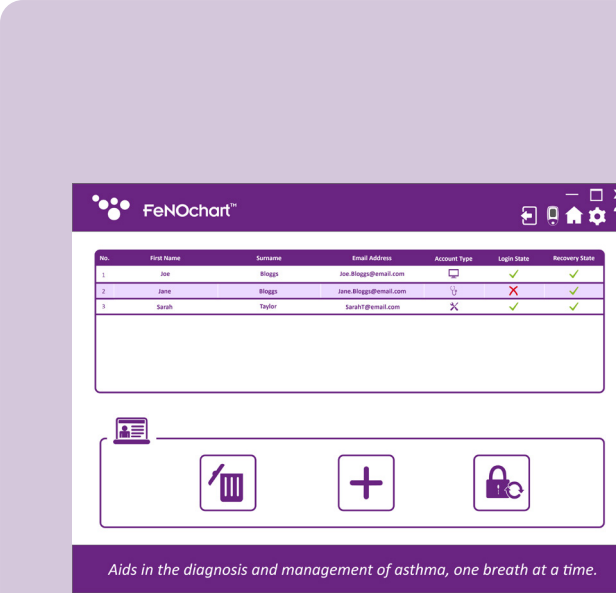
Next, enable synchronisation by changing 'No Operation' to 'Device to Database'.

Ensure the 'CREATE' function is selected in order to create a new profile within the FeNOchart™ database.

To complete the synchronisation, click the ‘Synchronisation’ button at the top right of the table. A pop-up will appear upon completion. The patient profile and readings will now exist on both the NObreath® FeNO device and in the FeNOchart™ database.

Replace results on an existing profile via synchronisation

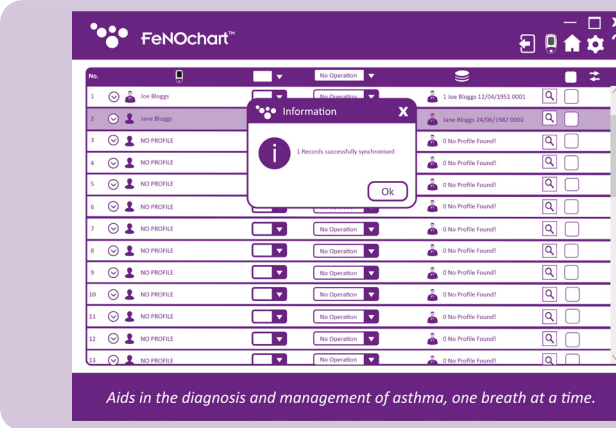
Select the patient to replace results with from the FeNOchart™ database.

NOTE: *The results from the suggested patient profile within the FeNOchart™ database will be overwritten by the results for the matching profile on the NObreath® device.*

Next, enable synchronisation by changing ‘No Operation’ to ‘Device to Database’.

Ensure the ‘REPLACE’ function is selected as this will overwrite the results within the matching profile from the FeNOchart™ database. Verify the suggested profile is the correct profile to clear the results from or search the database for another profile.

NOTE: *By checking the ‘Overwrite Profile Name’ box, the profile name in the FeNOchart™ database will also be overwritten to match the name as displayed on the device.*

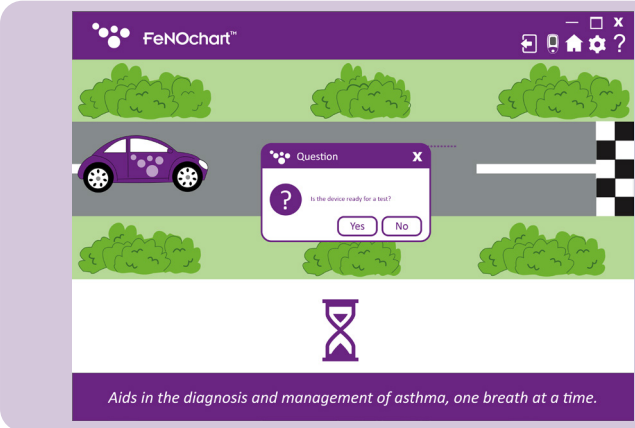
To complete the synchronisation, click the ‘Synchronisation’ button at the top right of the table. A pop-up will appear upon completion. The patient profile readings (and name if checked) will now have been overwritten on the FeNOchart™ database to match those of the corresponding profile on the NObreath® device.

How-To Videos

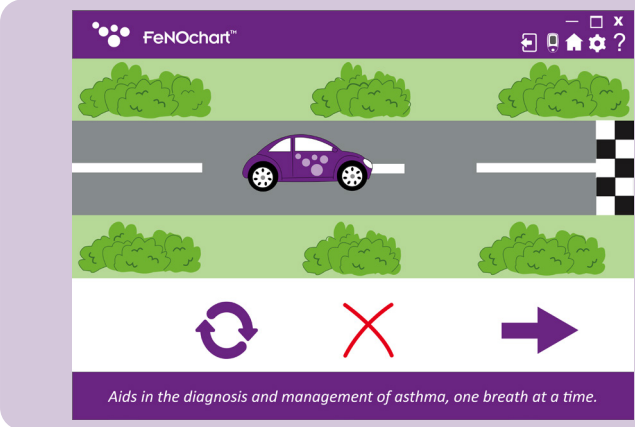
The NObreath® FeNO device ‘How-To’ videos are automatically included with the FeNOchart™ software. To see the list of videos, click on the ‘Play’ icon and select the desired video to play.

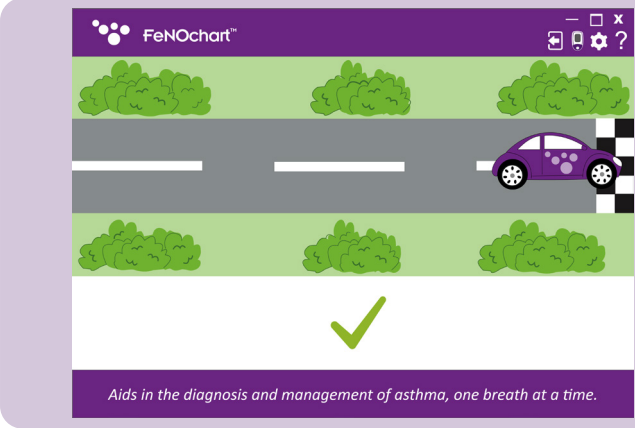
Taking a Breath Test

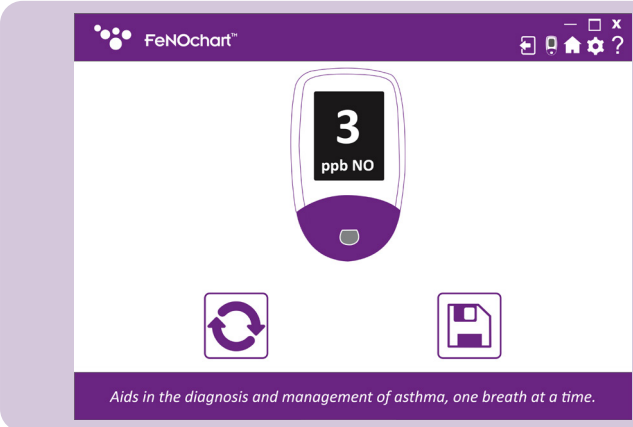
To take a breath test, click either the ‘Adult Breath Test’ or ‘Child Breath Test’ icon from the home screen. Follow the on-screen prompts to complete the breath test.

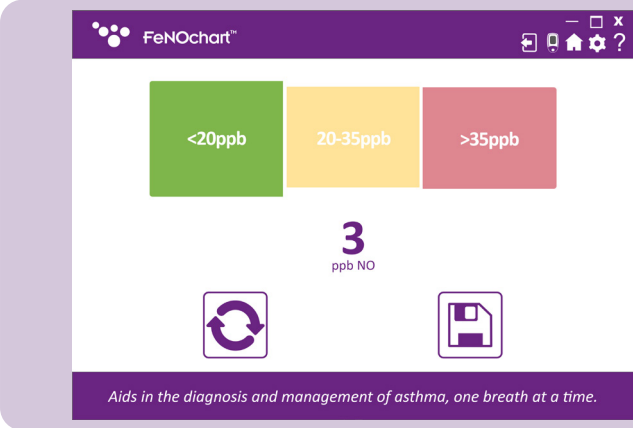
An unsuccessful breath test will show a red ‘x’.
To reattempt the breath test click the ‘Retry’ icon, or click the ‘Arrow’ to continue to the results.

A successful breath test will be shown by a green tick.

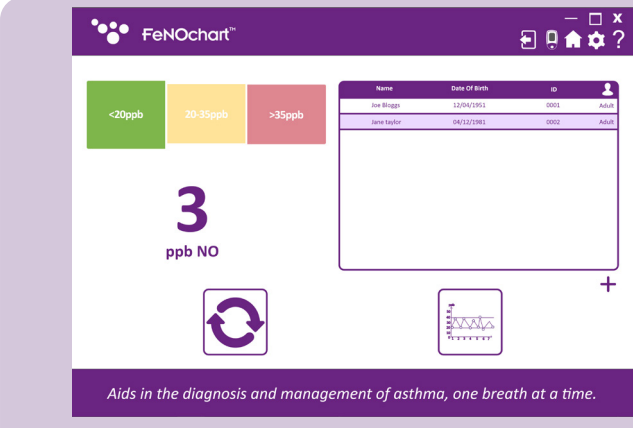
The result will then display on-screen. Click the 'Retry' icon to take a new test, or the 'Save' icon to save the result to a patient's profile.

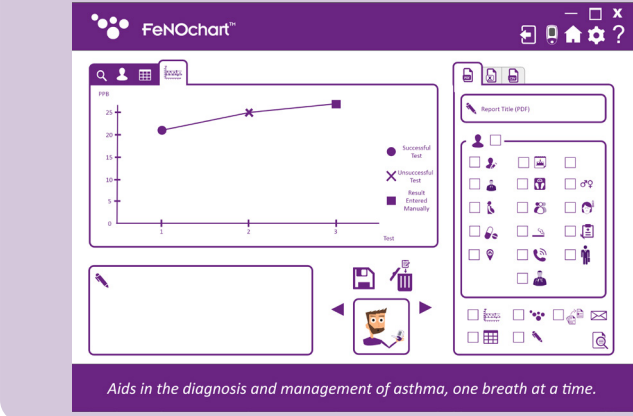
Select a patient from the list to save the result to their profile.

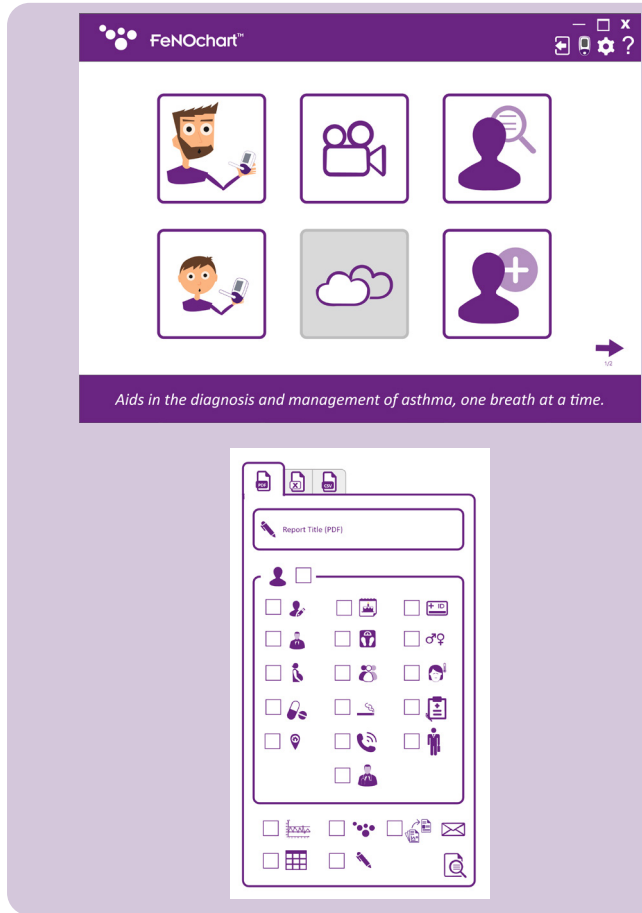
NOTE: *If taking an adult breath test, only adult patient profiles will be listed.*

Once the result has been saved to a patient's profile, another test can be taken by clicking the retry button, or click on the 'Graph' icon to see the graph of results within the patient's profile.

Generating Reports

Once a test has been completed and saved to a profile, a report can be generated. Select the patient profile required, then select the report criteria that is to be included.

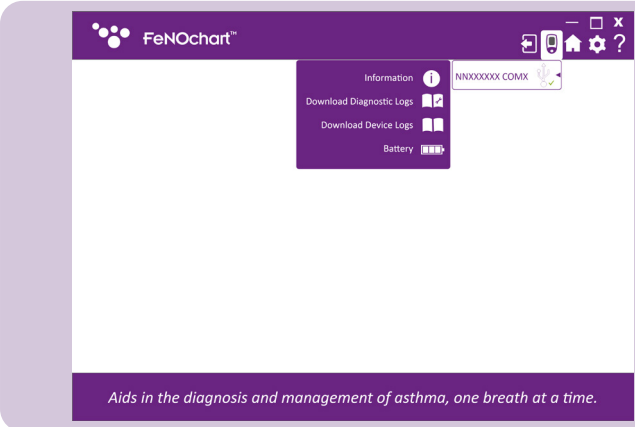
There are also options to include a graph of results, table of results, company logo and additional notes in the report.

There is also an option to generate a short report, which includes patient results only. If a short report is desired, select this option then either the envelope icon to generate a report in the form of an email, or the document viewer icon for a report in a PDF file format.

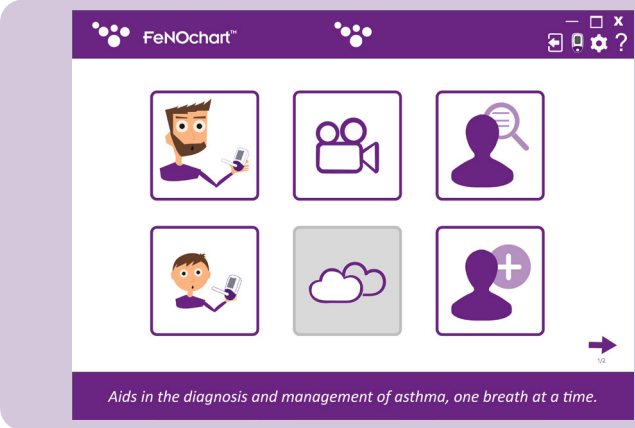
Ambient Testing

Coming Soon

Device Options

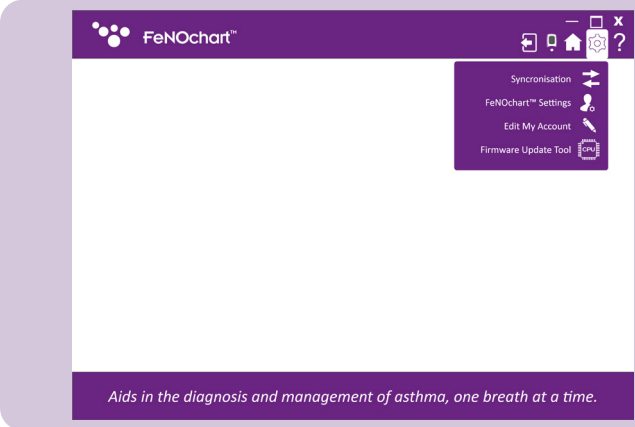
When a NObreath® device is connected to the FeNOchart™ software, the 'NObreath®' icon will appear in the top right-hand corner. To access these options, click the 'Device Options' icon at the top right of the window.

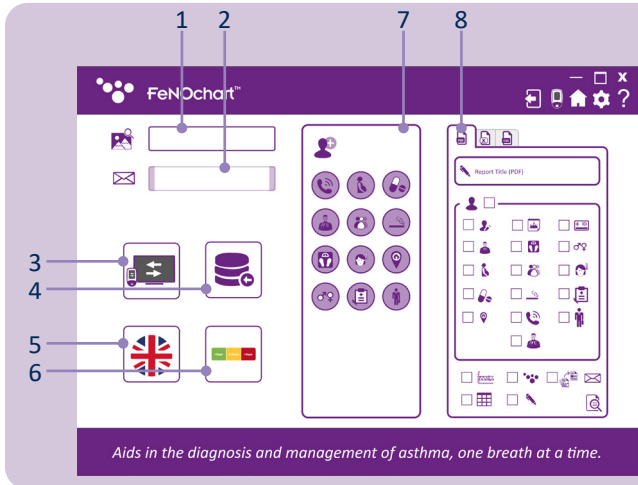
NOTE: When data is being transferred from the NObreath® to the computer, for example when downloading any logs from the Device Options menu, the Bedfont® logo will flash at the top of the window.

Software Settings

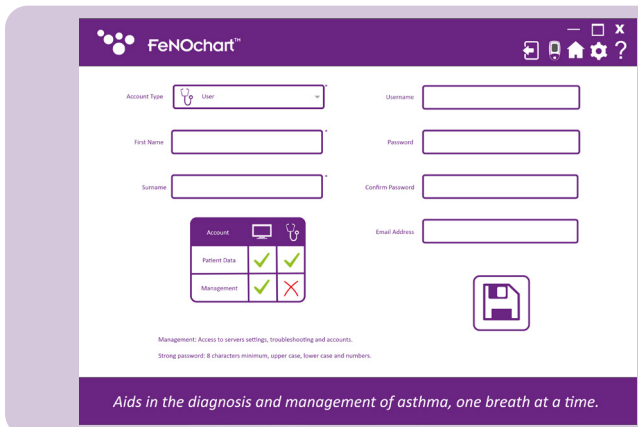
In the software settings, it is possible to change the default criteria for patient profiles and reports, and much more. To access these options, click the 'Settings' icon at the top right of the window.

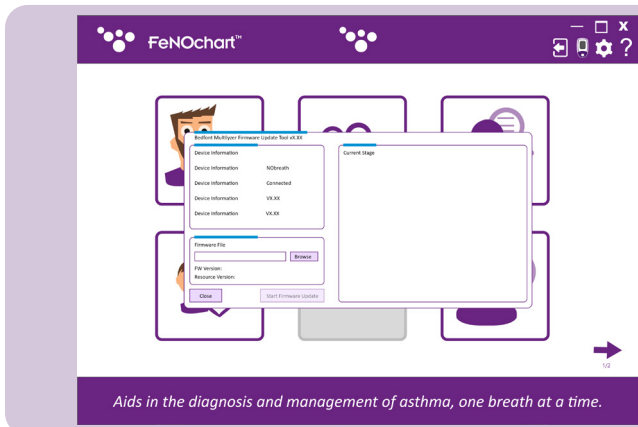
FeNOchart™ Settings

1. Set default report logo
2. Set default email address (coming soon)
3. Enable/disable screen mirroring (coming soon)
4. Move database (local or networked)
5. Change software language
6. Enable/disable results interpretation
7. Change default patient profile criteria
8. Change default report criteria

Edit User Account

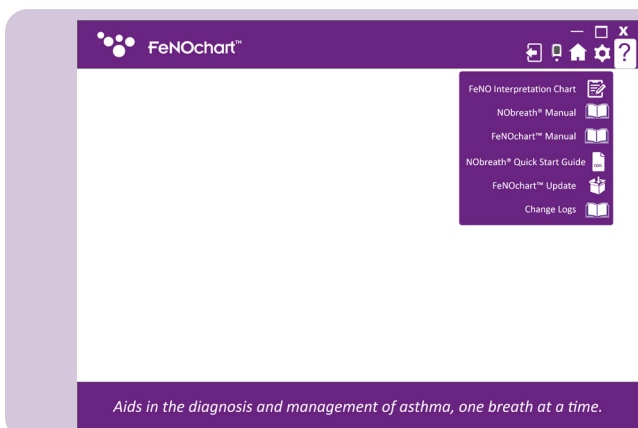
From here, users can change their password or security questions and answers.

Firmware Update Tool

When the NObreath® firmware needs updating, this can be done by choosing the Firmware Update Tool from the Software settings menu.

NOTE: In the first instance, the Firmware Update Tool must be installed by following the on-screen instructions. Each time after, selecting the Firmware Update Tool from the Software Settings menu will simply open the program.

Help Menu

The help menu can be accessed by clicking on the 'Question Mark' icon at the top right of the screen. From here, useful documents can be downloaded and FeNOchart™ can check for software updates.

Icons Explained

Patient name(s)		Gender		Profession	
Child profile		Medication		Prescriptions	
Date of Birth		Pregnancy status		Additional notes	
Telephone number		Smoking status		Next action	
Patient ID		Ethnicity		Previous action	
Doctor		Address		Save	
Adult profile		Symptoms		Delete patient profile	
Weight		Include table of results on report		View table of results	
Include graph on report		Include logo on report		Email report	
Adult breath test		Child breath test		View graph of results	
Preview report		View additional notes		View patient database	
Delete a result		View patient profile		Add a result manually	
Date		Time		Short report	

Responsible Manufacturer and Contacts

Bedfont® Scientific Ltd.
Station Yard, Station Road,
Harrietsham,
Maidstone, Kent,
ME17 1JA
United Kingdom

www.bedfont.com
www.nobreathfeno.com
ask@bedfont.com
0044 1622 851122

Bedfont® Scientific Limited reserves the right to change or update this literature without prior notice.
Registered in: England and Wales. Registered No: 1289798

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No.	First Name	Surname	Email Address	Account Type	Login State	Recovery State
1	Joe	Bloggs	Joe.Bloggs@email.com	👤	✓	✓
2	Jane	Bloggs	Jane.Bloggs@email.com	🔒	✗	✓
3	Sarah	Taylor	SarahT@email.com	✗	✓	✓

Information
The new credentials for this account are:
Username: Y0Y1QZUJ
Password: Y0Y1QZUJ

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No.	First Name	Surname	Email Address	Account Type	Login State	Recovery State
1	Joe	Bloggs	Joe.Bloggs@email.com	👤	✓	✓
2	Jane	Bloggs	Jane.Bloggs@email.com	👤	✓	✓
3	Sarah	Taylor	SarahT@email.com	✗	✓	✓

Question
Are you sure you want to delete Jane Bloggs (Jane.Bloggs@email.com)?
Yes No

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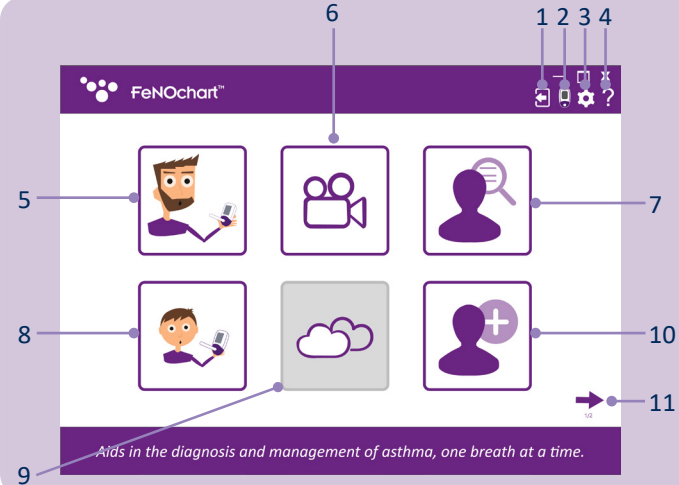
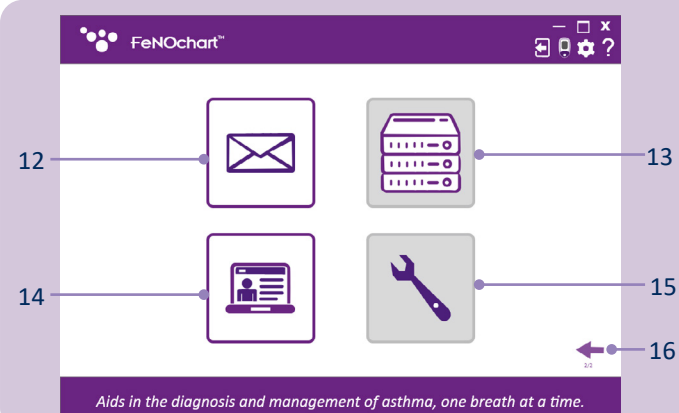
The screenshot shows the FeNOchart Home interface. At the top, there is a purple header with the FeNOchart logo and a navigation bar containing icons for mobile, settings, and help. Below the header, there are six main icons arranged in a 2x3 grid. Callout 1 points to the top right corner, 2 to the NObreath logo, 3 to the settings icon, 4 to the help icon, 5 to the adult test icon, 6 to the video demonstrative icon, 7 to the patient database icon, 8 to the children's test icon, 9 to the environment test icon, 10 to the add new patient icon, and 11 to the next page arrow. A footer at the bottom contains the text: "Aids in the diagnosis and management of asthma, one breath at a time."

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The screenshot shows the FeNOchart settings interface. It features a purple header with the FeNOchart logo and a navigation bar with icons for mobile, settings, and help. Below the header, there are four main icons arranged in a 2x2 grid. Callout 12 points to the email settings icon, 13 to the server settings icon, 14 to the administrator panel icon, 15 to the problem resolution icon, and 16 to the previous page arrow. A footer at the bottom contains the text: "Aids in the diagnosis and management of asthma, one breath at a time."

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No.	First Name	Surname	Email Address	Account Type	Login State	Recovery State
1	Joe	Bloggs	Joe.Bloggs@email.com	🔒	✓	✓
2	Jane	Bloggs	Jane.Bloggs@email.com	🔒	✗	✓
3	Sarah	Taylor	SarahT@email.com	✗	✓	✓

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No.	First Name	Surname	Email Address	Account Type	Login State	Recovery State
1	Joe	Bloggs	Joe.Bloggs@email.com	🔒	✓	✓
2	Jane	Bloggs	Jane.Bloggs@email.com	🔒	✓	✓
3	Sarah	Taylor	SarahT@email.com	✗	✓	✓

Question
Are you sure you want to delete Jane Bloggs (Jane.Bloggs@email.com)?

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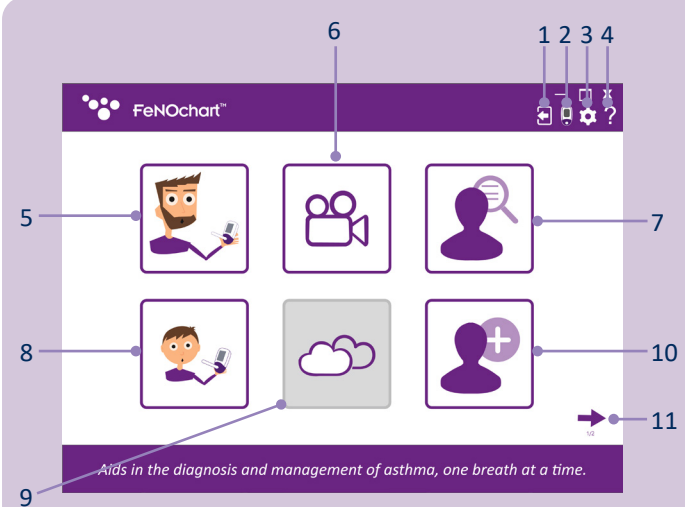
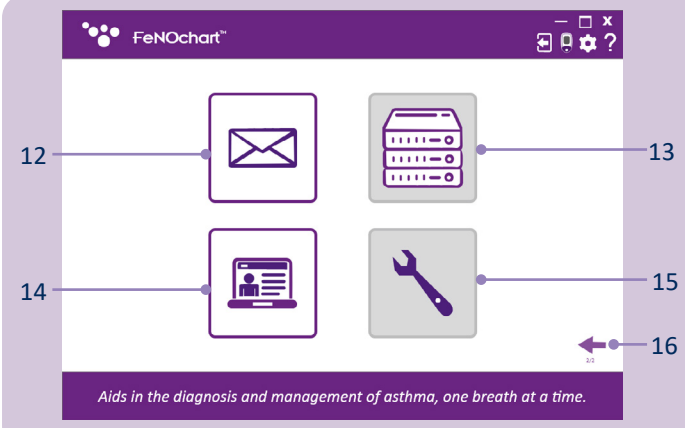
Gebruikersinterface

The screenshot shows the FeNOchart Home Screen. At the top, there is a purple header with the FeNOchart logo and a navigation bar containing icons for mobile, settings, and help. Below the header, there are six main icons arranged in a 2x3 grid. A bottom bar contains a navigation arrow and a slogan: "Aids in the diagnosis and management of asthma, one breath at a time." Numbered callouts point to various elements: 1 (Logout), 2 (NObreath®-options), 3 (Settings), 4 (Help), 5 (Adult test), 6 (Demo videos), 7 (Patient database), 8 (Child test), 9 (Environment test), 10 (Add new patient), and 11 (Previous page).

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The screenshot shows the FeNOchart Admin Panel. It features a purple header with the FeNOchart logo and a navigation bar with icons for mobile, settings, and help. The main area contains four icons: an envelope, a server rack, a laptop with a user icon, and a wrench. A bottom bar contains a navigation arrow and the same slogan as the Home Screen: "Aids in the diagnosis and management of asthma, one breath at a time." Numbered callouts point to: 12 (Email settings), 13 (Server settings), 14 (Admin panel), 15 (Problem solving), and 16 (Previous page).

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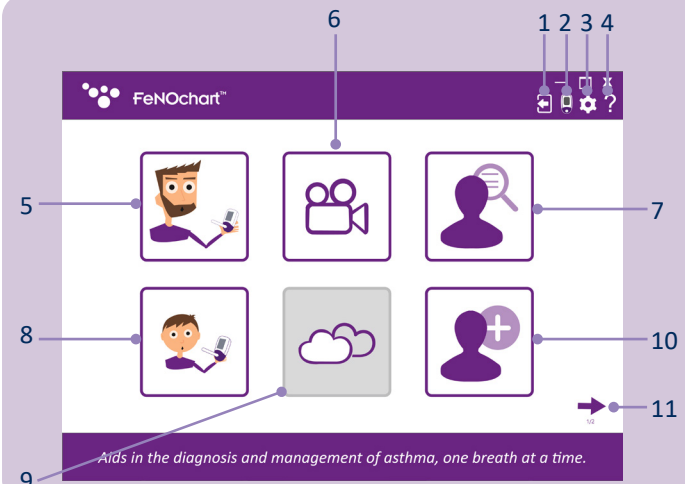
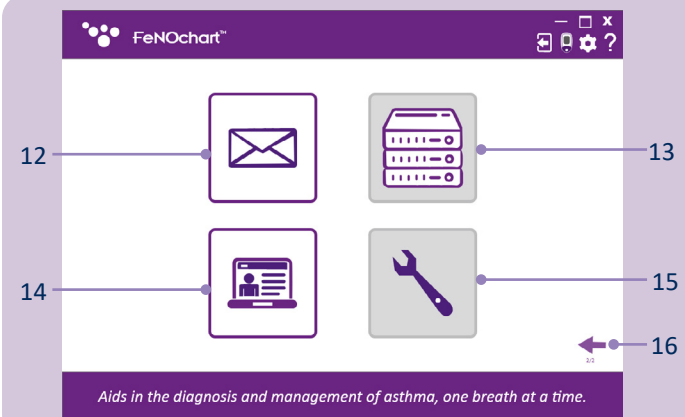
The screenshot shows the main menu of the FeNOchart application. At the top, there is a purple header with the FeNOchart logo and four icons: a smartphone, a gear, a question mark, and a magnifying glass. Below the header, there are six large square buttons arranged in a 2x3 grid. The buttons contain icons for: a person with a smartphone (5), a video camera (6), a person with a magnifying glass (7), a person with a smartphone (8), a cloud with a plus sign (9), and a person with a plus sign (10). At the bottom, there is a purple footer with the text "Aids in the diagnosis and management of asthma, one breath at a time." and a right-pointing arrow (11).

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The screenshot shows the settings menu of the FeNOchart application. At the top, there is a purple header with the FeNOchart logo and three icons: a minus sign, a square, and an 'x'. Below the header, there are four large square buttons arranged in a 2x2 grid. The buttons contain icons for: an envelope (12), a server rack (13), a laptop with a person icon (14), and a wrench (15). At the bottom, there is a purple footer with the text "Aids in the diagnosis and management of asthma, one breath at a time." and a left-pointing arrow (16).

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A screenshot of the user account creation form in the FeNOchart application. The form is on a purple background with white text and input fields. It includes fields for Account Type (set to Admin), Username, First Name, Surname, Password, Confirm Password, and Email Address. There is a '저장' (Save) icon at the bottom right. A small table with account permissions is visible on the left side of the form. Below the form, there is a dark purple banner with the text "Aids in the diagnosis and management of asthma, one breath at a time."

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Account Type: User
Username:
First Name:
Surname:
Password:
Confirm Password:
Email Address:

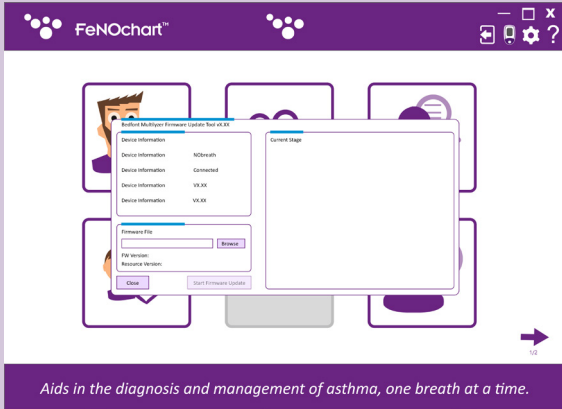
Account	Device	User
Patient Data	✓	✓
Management	✓	✗

Management: Access to servers settings, troubleshooting and accounts.
Strong password: 8 characters minimum, upper case, lower case and numbers.

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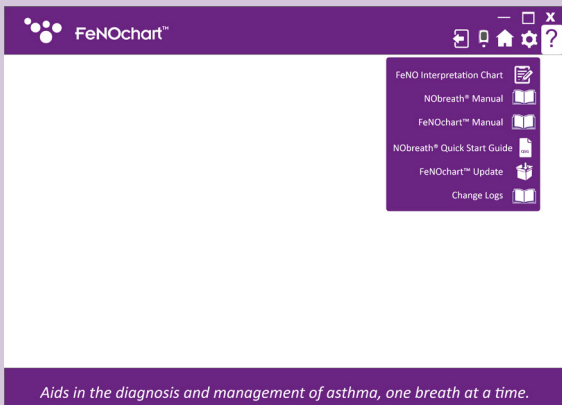
© Bedfont[®] Scientific Limited 2026

Issue 11 - February 2026, Part No: LAB776

Bedfont[®] Scientific Limited reserves the right to change or update this literature without prior notice.
Registered in: England and Wales. Registered No: 1289798

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Bedfont® Scientific Limited, bu literatürü önceden bildirim yapmaksızın değiştirme veya güncelleme hakkını saklı tutar.
Bedfont[®] Scientific Ltd.
Emergo Europe B.V.