Smokerlyzer® Range

For use with piCO™, piCO^{baby}™ and Micro⁺™

Quick Start Guide

CE
2797

Download
your **FREE**
patient management
software
COdata⁺™
at...
www.bedfont.com

*Helping people to stop smoking,
one breath at a time.*

bedfont
est. 1976

Contents

English (EN)	4
Français (FR)	5
Deutsch (DE).....	6
Español (ES)	7
Italiano (IT)	8
Português (PT).....	9
Nederlands (NL).....	10
Polski (PL)	11
Svenska (SE).....	12
հայերէն (AM)	13
العربية (AR)	14
български (BG)	15
Eský (CZ)	16
Dansk (DK).....	17
Suomi (FI)	18
Γραικός (GR)	19
Magyar (HU)	20
日本 (JP).....	21

Lietuvių (LT)	22
Norsk (NO).....	23
România (RO)	24
Русский (RU)	25
Slovenský (SK).....	26

Smokerlyzer[®]

For use with piCO™, piCO^{baby}™ and Micro⁺™

Taking a Breath Test

1. Attach a breath sampling D-piece™ and new SteriBreath™ Eco mouthpiece.
2. Turn on the device by pressing the power button once.
3. Press 'Breath Test' symbol on screen, .
4. To cancel the breath test, press .
5. Inhale and hold breath for the pre-set 15 second countdown.
6. A beep will sound during the last 3 seconds of the countdown.
7. Blow slowly into mouthpiece, aiming to empty the lungs completely.
8. The ppm and equivalent %COHb and/or %FCOHb levels will rise and hold on-screen.
9. On the piCO™ and piCO^{baby}™, when the test is finished   will appear at the bottom of the screen.
10. On the Micro⁺™, when the test is finished    will appear at the bottom of the screen.
11. If a high reading has been recorded, you can mute the sounder by pressing .
12. To repeat breath test, press  once to return to the home screen and repeat steps 3-8.
13. To save the reading (Micro⁺™ only) press  and select the relevant patient profile.
14. Remove the D-piece™ between tests to purge sensor with fresh air.
15. To switch off, press and hold the power button for 3 seconds, unit will also power off after 2 minutes of inactivity to save power.

To download the full manual and COdata⁺™ please go to www.bedfont.com

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© Bedfont[®] Scientific Limited 2025
Bedfont[®] Scientific Ltd.
Emergo Europe B.V.
Bedfont[®] Scientific Limited reserves the right to change or update this literature without prior notice.