# Gastro<sup>+</sup> Gastrolyzer<sup>®</sup>

## Quick Start Guide

*Helping to detect gastrointestinal disorders,  
one breath at a time.*

CE  
2797

Download  
your **FREE**  
patient management  
software  
**GastroCHART™**  
at...  
[www.bedfont.com](http://www.bedfont.com)

**Rx**only

# Contents

English (EN) .....	5
Français (FR) .....	6
Deutsch (DE).....	7
Español (ES).....	8
Italiano (IT) .....	9
Nederlands (NL).....	10
Polski (PL) .....	11
Português (PT).....	12
Svenska (SE).....	13
հայերէն (AM) .....	14
български (BG) .....	15
Eský (CZ) .....	16
Dansk (DK) .....	17
Suomi (FI) .....	18
Γραικός (GR) .....	19
Hrvat (HR).....	20
Magyar (HU) .....	21
日本 (JP).....	22

Lietuvių (LT) .....	23
Norsk (NO).....	24
Português Brasileiro (PTBR).....	25
România (RO) .....	26
Русский (RU) .....	27
Slovenski (SI).....	28
Slovenský (SK).....	29
Türkçe (TR).....	30
Україна (UA).....	31

EN	Gastrolyzer® breath Hydrogen device.
FR	Appareil d'hydrogène, respiratoires Gastrolyzer®.
DE	Gastrolyzer® Gerät für Atemwasserstoff.
ES	Dispositivo de hidrógeno en aliento Gastrolyzer®.
IT	Dispositivo di idrogeno, nel respiro del Gastrolyzer®.
NL	Gastrolyzer®-apparaat voor waterstof.
PL	Urządzenie Gastrolyzer® monitorujące poziom wodoru, wydychanym powietrzem.
PT	Dispositivo de Hidrogénio do Sopro Gastrolyzer®.
SE	Gastrolyzer® enhet av vätgas i utandningsluften.
AM	Gastrolyzer® սարք՝ արտաշնչված շնչում ջրածին չափելու համար.
BG	Уред Gastrolyzer® за измерване на водород издишания въздух.
CZ	Přístroj Gastrolyzer® k měření vodíku vydechovaném vzduchu.
DK	Gastrolyzer®-apparat til måling af hydrogen i udåndingsluften.
FI	Gastrolyzer®-laite mittaamaan vetyä uloshengitetyssä ilmassa
GR	Συσκευή Gastrolyzer® για τη μέτρηση υδρογόνου στην εκπνεόμενη αναπνοή.
HR	Gastrolyzer® uređaj za mjerenje vodika u izdahnutom zraku.
HU	A Gastrolyzer® készülék hidrogén mérésére a kilélegzett légzésben.
JP	Gastrolyzer®—呼気中の水素濃度を測定する装置
LT	„Gastrolyzer®“ prietaisas vandeniliui iškvėptame ore matuoti.
NO	Gastrolyzer®-enhet for måling av hydrogen utåndingsluften.
PTBR	Dispositivo Gastrolyzer® para medir hidrogênio a respiração exalada.
RO	Dispozitiv Gastrolyzer® pentru măsurarea hidrogenului în respirația expirată.
RU	Gastrolyzer® — Устройство Gastrolyzer® для измерения содержания водорода в выдыхаемом воздухе
SI	Naprava Gastrolyzer® za merjenje vodika v izdihanem zraku.
SK	Přístroj Gastrolyzer® na meranie vodíka vo vydychovanom vzduchu.
TR	Gastrolyzer® cihazı, solunan havadaki hidrojen ölçmek için kullanılır.
UA	Прилад Gastrolyzer® для вимірювання вмісту водню у видихуваному повітрі

## Taking a Breath Test

1. Attach a breath sampling D-piece™ and a new SteriBreath™ Eco mouthpiece.
2. Turn on the device by pressing the power button once.
3. Press 'Breath Test' symbol on screen .
4. To cancel the breath test, press .
5. Inhale and hold breath for the pre-set 15 second countdown.
6. A beep will sound during the last 3 seconds of the countdown.
7. Blow slowly into mouthpiece, aiming to empty the lungs completely.
8. The ppm levels will rise and hold on-screen.
9. When the test is finished    will appear at the bottom of the screen.
10. To repeat the breath test, press  and repeat steps 4-8.
11. To return to the home screen, press .
12. To save the reading, press  and select the relevant patient profile.
13. Remove the D-piece™ between tests to purge sensor with fresh air.
14. To switch off, press and hold the power button for 3 seconds, device will also power off after 45 minutes of inactivity to save power.

## Taking a Facemask Test

1. Attach a new facemask and face mask sampling system.
2. Turn on the device by pressing the power button once.
3. Press .
4. To cancel the test, press .
5. Inhale and exhale in to the face mask, the device will take the reading in real-time.
6. The ppm levels will rise and hold at the peak level, when this has stopped rising for a period of 5 consecutive seconds the test result can be recorded and the test stopped.
7. When the test is finished    will appear at the bottom of the screen.
8. To repeat the test, press  once to return to the home screen and repeat steps 3-6.
9. Remove the face mask sampling system between tests to purge sensor with fresh air.
10. To switch off, press and hold the power button for 3 seconds, device will also power off after 45 minutes of inactivity to save power.

To download the full Gastro™ manual and GASTROCHART™ please go to [www.bedfont.com](http://www.bedfont.com)

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© Bedfont® Scientific Limited 2026
**Bedfont® Scientific Ltd.**
**Emergo Europe B.V.**
Bedfont® Scientific Limited reserves the right to change or update this literature without prior notice.